

ApoWIPE®

Disinfection wipe for cleaning vascular catheters



Reduce infections

ApoWIPE Ethanol 80% is suitable for disinfecting injection ports, caps and various types of connectors, as well as local skin disinfection.

The disinfection wipe is **individually wrapped** to ensure its purity in every situation. Size of the package is also designed so that the wipe may be taken out from the packaging aseptically.

The **size** of the wipe is designed so that the wipe can be handled hygienically and touching of the disinfected area can be avoided.

The **rough wipe material** ensures efficient mechanical cleaning.

The **thickness of the wipe** ensures adequate moisture and thus the survival of ethanol as 80% throughout the procedure.




By using of ApoWIPE disinfection wipe the aseptic handling of vascular catheters can be enhanced. With better aseptic a number of significant benefits can be achieved:

- less infections
- cost savings
- improved patient safety

Item	Description	Wipes per carton
021871	ApoWIPE IV Port Disinfection	800

ApoWIPE disinfection wipe for cleaning vascular catheters

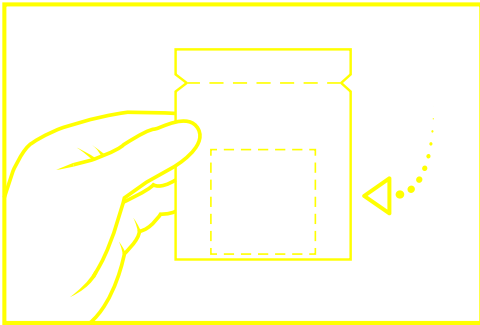

1. The wipe is placed in the bottom of the package so that it remains as clean as possible when opening the package.




2. Open the pouch directly and snugly along the tear line so that the wipe will remain untouched.



3. Take the wipe out of the package.



4. Scrub the hub by rubbing it with the wipe thoroughly for 15 seconds.

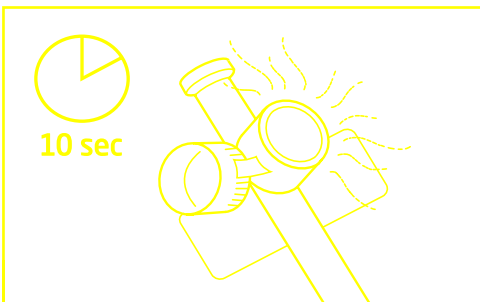

5. Let the hub dry for 10 seconds.