

ApoWipe®

ApoWIPE Surface Disinfection - Ethanol 80%

ApoWIPE is a product series from the Danish company Apodan A/S.

ApoWIPE Ethanol 80% Disinfection is used to disinfect all surfaces able to withstand ethanol.

Usage

ApoWIPE Ethanol 80% Disinfection is suitable for areas with frequent manual contact, sanitary equipment, beds, computers and apparatus able to withstand ethanol. Should not be used on lacquered surfaces.

Disinfects effectively to prevent e.g. multi-resistant bacteria, MRSA, E.coli, mycoviruses and most viruses, including norovirus.

Use on a clean and dry surface. Use the wipe to keep the surface wet for at least 1 min.

One wipe has a disinfection range of:
Regular 1-2 m² / Large 2-4 m²

The product is ready-to-use and comes in two sizes.

HAZARD LABELS

H225 Highly flammable liquid and vapour.
H319 Causes serious eye irritation.
P210 Keep away from heat/sparks/open flames/hot surfaces — No smoking.
P280 Wear protective gloves.



Ingredients:

Væske:

Aqua, Ethanol, Denatonium benzoate

Serviet:

Nonwoven spunlaced, Viscose/Polyester

Foil:

PET / PE

One wipe can disinfect:

1-2 m²

Item nr.	Item name	Size of wipe	Wipes per pack	Packs per carton
021870	ApoWIPE Ethanol 80%, Regular	30 x 22 cm	32	12

For further information, please contact:
Tel.: 32 97 15 25 | www.apodan.dk | info@apodan.dk